Bellman Scart

Loop amplifier

model BE1140

Bellman & Symfon

Bellman Scart Loop amplifier model BE1140

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User manual, Bellman Scart loop amplifier

The Bellman Scart loop amplifier model BE1140 forms a part of the system Bellman & Symfon has for TV listening. Simply connect the Bellman Loop cushion model BE1150 (or a neck loop) to the loop amplifier. You can then listen to the TV with your hearing aid or your Bellman Response model BE1040 personal amplifier set to the "T" position.*

**You can connect a neck loop, a pair of headphones or an extra loudspeaker to the loop amplifier but these are not part of our product range. Contact your local Bellman & Symfon sales representative for more information. Please observe that the headphones and loudspeaker may only be connected using an adaptor lead.*

This is what you do!

- 1. Connect the Bellman Scart loop amplifier to the SCART connector on the rear of the TV, and connect the accompanying mains adaptor to an ordinary mains socket.**
- 2. Connect the chair loop, neck loop or headphones to the connector on the loop amplifier. Put the chair loop on the chair, or put the neck loop around your neck.**
- 3. If you use the chair loop or neck loop, adjust the volume of the hearing aid when it is in the "T" mode. If you use headphones, adjust the volume in them by turning the volume control on the headphones.**
- 4. If necessary, the volume can also be adjusted via the volume control on the Bellman Scart loop amplifier.**

Settings

The volume can be adjusted as necessary, using the volume control on the Bellman Scart loop amplifier.

General

The Bellman chair loop or a neck loop must be connected to the Bellman Scart loop amplifier to allow other hearing appliances with built-in loop coil (such as hearing aids) to receive the amplified signal.

Fault finding

Faults are not always serious. Please follow the fault-finding chart below before you call for service.

Symptom	Action
The Bellman Scart loop amplifier does not work.	<ol style="list-style-type: none">1. Check that the power supply is plugged into the wall socket.2. Check that the seat loop or neck loop is securely connected to the connector on the Bellman Scart loop amplifier.3. If the Bellman Scart loop amplifier is used with a hearing aid, check that the volume is turned up and that it is set to "T" mode.4. Check that the volume on the loop amplifier is turned up.
The TV does not have a SCART connector, or the SCART connector is occupied.	Ask the nearest TV/Radio dealer for advice.

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