MCS ST-100

- EN Hard of hearing telephone
- NL Telefoon voor slechthorenden
- DE Telefon für Schwerhörigen
- FR Téléphone pour malentendants

EN

4 - 12

NL

13 - 22

DE

23 - 33

FR

34 - 43

TABLE OF CONTENTS

Package content	4
Product features	4
Location and functions of controls	5
Installation	6
Quick operation guide	7
Function setup	8
Operation	9
Trouble shooting guide	11
Important safety instructions	11
Wall mounting	12
Wall mounting the phone	12
Service and guarantee	12


Package content

- 1 x base station
- 1 x handset
- 1 x curl cord
- 1 x line cord
- 1 x phone plug
- 1 x manual

PRODUCT FEATURES

1. Battery free
2. Commercial grade high-quality speakerphone
3. LCD displays phone numbers, date and time
4. Incoming call indication
5. Speakerphone auto answering launched while receiving calls from 5 preset caller numbers
6. Hold with mute, music and extension release
7. 16 dialled numbers review and redial
8. Automatic call timer
9. Intelligent auto redial function
10. Flash time adjustable 100 ms, 200 ms, 300 ms, 400ms, 600 ms, 900 ms
11. 80 phone book entries
12. One-touch emergency keys with auto-redial function
13. 16 melodies selectable
14. Ringer volume 8 levels adjustable
15. FSK/DTMF available
16. Call waiting caller ID
17. 76 caller ID memories
18. Multiple language support
19. Hearing Aid compatible
20. Cellular phone RFI noise immunity


LOCATION AND FUNCTIONS OF CONTROLS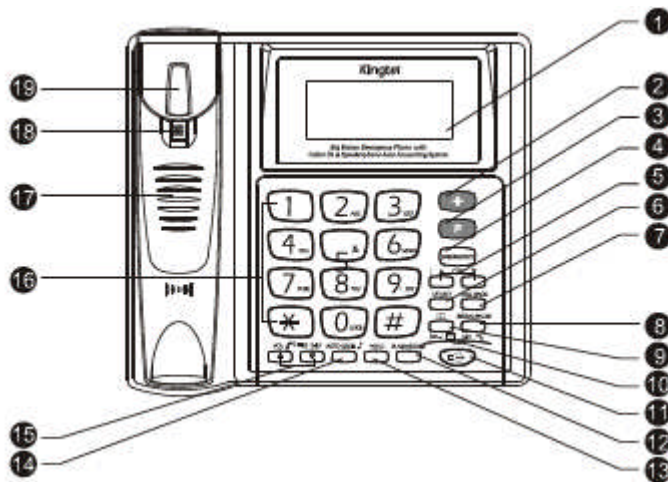

- | | |
|-----------------------------|---|
| 1. LCD display | Displays the phone number, date and time and other information. |
| 2. + | Key can store one phone number for speed dialing |
| 3. P | Key can store one phone number for speed dialing |
| 4. Emergency | Key can store emergency phone number as for speed dialing |
| 5. CLID ▲ en ▼ | Review caller ID's. Also can be uses within the menu to select telephone settings |
| 6. Del/Set | a. Press and hold for 3 seconds to bring up setup menu (In on-hook status)
b. Press to delete or edit the telephone number when editing phone book or viewing call ID's. |
| 7. Call Back/Clock | a. Press to dial the stored number when viewing caller ID's or phone book
b. Press and hold for 3 seconds to set alarm |
| 8. Redial/Pause | a. Press repeatedly to review the outgoing numbers (on-hook)
b. Press to dial the last number (off-hook)
c. Press for a break off 1,2/2.4/3.6 seconds |
| 9. ☐ | Press to enter the phone book operation |
| 10. ringing indicator | Lights up when the unit is in use, flash when there is a new call, flash when on hold |
| 11. 📞 | Press for handsfree dialing and speak without lifting the handset |
| 12. Flash/store | a. for PBX
b. Press to store phone numbers |
| 13. Hold | Press to hold the line. Turns sounds off. |
| 14. AUTO RD/🎵 | a. Press to auto-redial the last dialed number repeatedly until the phone is answered
b. Press and hold for 3 seconds to change the ringtone (16 melodies) |
| 15. 🔊🔊🔊🔊🔊 | a. Press and hold to adjust ringer volume (on-hook)
b. Press to adjust handset volume (off-hook)
c. Press to adjust speaker volume (speakerphone mode) |
| 16. Dialling keys (0-9,*,#) | Dial numbers and extensions |
| 17. Speaker | Ringing sound is spread from here |
| 18. Wall mounting hook | Turn 180 degrees to hang up the handset when the unit is mounted on the wall |
| 19. Hook on/off switch | Press to end a call |

INSTALLATION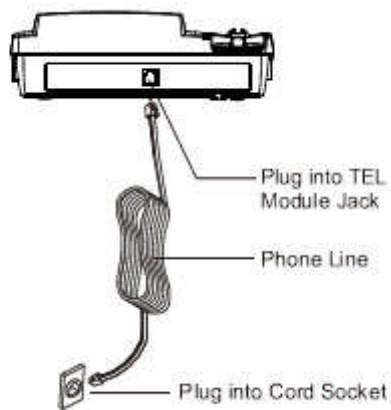

1. Open the battery compartment cover and load 4 AA batteries (not included).
2. Connect one end of coil cord to the handset jack and connect the other end to the jack of the base unit.
3. Plug one end of phone line to line jack in the base unit and connect the other end to the wall jack

NOTE

1. The phone works well with the line power alone. Batteries are only needed for the clock to function.
2. Use only the following type and size batteries: AA cellUm3. The proper polarity or direction must be observed.
3. Please change your batteries every 6 months
4. All batteries should be changed at once.
5. Please pull out the line cord before replacing batteries

QUICK OPERATION GUIDE

1. Make a call:
 - a. Pick up the handset or
 - b. press op  -> dial number
2. Answer a call:
 - a. Pick up the handset or
 - b. press 
3. End a call:
 - a. Hang up the handset or
 - b. press 
4. Redial:
 - a. On-hook; press **REDIAL/PAUSE** repeatedly to review dialed numbers, then pick up the handset or press  or press **CALL BACK**
 - b. Off-hook; press **REDIAL/PAUSE** for the last number
5. Pre-dial :

On-hook, dial number, then pick up handset
6. Auto redial:

On-hook, press **AUTO RD/▲♪**
7. Caller ID review
On-hook; press **CLID▲or▼**
8. Delete caller ID:
 - a. On-hook; press **CLID▲or▼** -> select a call -> press **DEL/SET**
 - b. Off-hook; press **CLID▲or▼** -> press **DEL/SET** 3 seconds to delete all ID's
9. Call back from caller ID:

On-hook; press **CLID▲or▼** -> select a call -> press **CALL BACK** or 
10. Set emergency speed dial numbers
On-hook; enter phone number -> press **FLASH/STORE** , then press **+** or **P** or **EMERGENT**, press #
11. Emergency speed-dial :

Press **+** or **P** or **EMERGENT**
12. Store number to phone book:

On-hook; press , then press ▲ enter name, press , enter phone number, press 
13. Dial from phone book:
 - a. On-hook; press , press ▼ , select a number, press **CALL BACK** , or pick up handset or press 
 - b. Off-hook; press , press ▼ , select a number, press **CALL BACK**

FUNCTION SETUP

1. Set the language:
In on-hook status, press and hold **DEL/SET** for 3 seconds to enter LANGUAGE SET UP. Press **CLID▲or▼** to select the language. Press **DEL/SET** to confirm.
2. Set the contrast:
After setting LANGUAGE, press **CLID▲or▼** to select the contrast (1-5). Press **DEL/SET** to confirm.
3. Set Year, Date and Time:
After setting the LCD contrast, the digits of the year will flash. Press **CLID▲or▼** to select the year, press **DEL/SET** to confirm. The digits of the month will flash, press **CLID▲or▼** to select the month and press **DEL/SET** to confirm. Perform the same approach to set DATE and TIME.
4. Set local code:
After setting the YEAR, DATE and TIME, the first digit of LOCAL AREA CODE will flash, press **CLID▲or▼** to select the digit and press **DEL/SET** to confirm. If you set an area code, ex. 248-1234567, when you receive a telephone number with area code 248, only 1234567 will show on the LCD.
5. Set Prefix Codes:
After setting the LOCAL CODE, the first of the three codes will flash. Press **CLID▲or▼** to select digit and press **DEL/SET** to save the selection. Use the same method to continue setting each code.
6. Set the ring melodies:
After setting the PREFIX CODE, press **CLID▲or▼** to select the melody (1-16) and press **DEL/SET** to confirm.
7. Set the ring volume:
After setting the RING MELODIES, press **CLID▲or▼** to select the ring volume (1-8). Press **DEL/SET** to confirm.
8. Set Flash time:
After setting the RING VOLUME, press **CLID▲or▼** to select the flash time (100ms, 200 ms, 300 ms, 400 ms, 600 ms, 900 ms) and press **DEL/SET** to confirm.
9. Set Pause time:
After setting FLASH TIME, press **CLID▲or▼** to select the pause time (1.2s, 2.4s, 3.6s) and press **DEL/SET** to confirm.
10. Set DTMF or Pulse mode:
After setting PAUSE TIME, press **CLID▲or▼** to select the mode (DTMF or PULSE) and press **DEL/SET** to confirm and complete the whole setting.

OPERATION

1. Make a call:

Pick up the handset or press  to get dial tone and dial the number. Use **DEL/SET** to edit the number. When the called party answers the phone, the LCS will time the call automatically.

2. Answer a call:

Pick up the handset or press  to answer the call.

3. End a call:

Hang up the handset or press  to end the call.

4. Switch talk mode from handset to speakerphone:

press  and then hang up the handset to talk through the speakerphone.

5. Switch talk mode from speakerphone to handset:

Pick up the handset and the talk mode will switch to handset automatically.

6. Hold:

Press **HOLD** and the call will be held. The handset can be hung up.

7. Hold release:

pick up the handset or press  or press **HOLD** button again to release a call.

8. Flash:

This function is mainly for PBX systems, it only works while call waiting service is provided by your local phone company.

- Press **FLASH** for call waiting service from your phone company
- Press **FLASH** to transfer the call to another extension if the phone runs on a PBX system.

9. Pre-dial:

In on-hook status; dial the phone number and then pick up the handset, the number will be dialed out automatically.

10. Pause:

Press **REDIAL/PAUSE** for a break of 1.2/2.4/3.6 seconds.

11. Redial:

- In on-hook status, press **REDIAL/PAUSE** repeatedly to review the last 16 dialed numbers, choose desired number and pick up the handset or press **CALL BACK** to dial out automatically.
- In off-hook status, press **REDIAL/PAUSE** to dial the last number.

12. Auto Redial:

In on-hook status, press **AUTO RD/** to auto redial the last dialed number repeatedly until the phone is answered.

13. 3 sets of one-touch emergency speed dial:

- Store:** In on-hook status, enter phone number, press **FLASH/STORE** and then press + (or P or EMERGENCY) to assign preferred speed dial buttons, press # to complete the setting.
- Dial:** Simply click on + (or P or EMERGENCY) button, whether it is on-hook or off-hook.

14. 5 sets speakerphone auto answering

In on-hook status; enter phone number, press **FLASH/STORE** and then press 1 (2 to 5) to store auto answering numbers, press # to complete the setting.

15. Alarm:

- Set alarm:** press **CALL BACK** for 3 seconds then enter alarm set-up. Press **CLID** or  to set hour, press **CALL BACK** to confirm and press **CLID** or  to set minute, press **CALL BACK** to complete the setting.
- Press **CALL BACK** to turn alarm on or off.

REMARK

The ST-100 has a built-in T-coil in the handset. In order to use make a call with the T-coil, put the hearing aid in T-position and hold the speakerpart of the handset behind the hearing-aid.

CALLER ID

1. Caller ID review:

In on-hook status, press **CLID▲or▼** to review the callers phone number and call in time.

2. Delete caller ID:

a. When checking incoming calls, press **CLID▲or▼** to search the call you would like to delete. Press **DEL/SET** once quickly to delete a single call.

b. When checking incoming calls, press **DEL/SET** for 3 seconds to delete all calls.

3. Call back from the caller ID:

In on-hook status, press **CLID▲or▼** to select the call you wish to dial from the caller IDs, press **CALL BACK** or  or pick up the handset to dial out.

PHONE BOOK

1. Store phone numbers:

In on-hook status, press **□□**, and the LCD will display **ADD ↑ SEARCH ↓**

a. Press **CLID▲** to add new information, enter the name and then press **□□** to access the telephone number input page; enter the telephone number, press **□□** to complete.

b. Perform the same approach to store more names and numbers to a maximum capacity of 80. Once reaching 80, **MEMORY FULL** will display on LCD.

2. Review phone book:

a. Press **□□** to enter the phone book page. Press **CLID▼** and LCD will display "PLS Input Name". Enter the name to view the telephone number.

b. Press **CLID▲or▼** to view the phone book.

c. Alphabetical searching is supported. Enter a letter to view the phone book by the first letter of the name.

3. Delete:

When viewing caller information in phone book, press and hold **DEL/SET** during 3 seconds to delete the name and number.

4. Dial from phone book:

a. In on-hook status, searching desired number in phone book, press **CALL BACK** or  or pick up the handset to dial out automatically.

b. In off-hook status, searching desired number in phone book, press **CALL BACK** to make a call and speak through handset.

TOUBLE SHOOTING GUIDE

Problem	Solution
Phone does not ring.	Make sure phone line cord is connected to bas unit and wall jack.
Ringer volume is too low.	Check if ringer volume is set on low (see page 4 for ringer adjustment).
Phone receives but cannot make calls.	Is the dial mode in accordance with the local telephone company.
Message waiting light does not work.	Check if the message waiting setting of base unit is in accordance with the setting of your local PABX system.

IMPORTANT SAFETY INSTRUCTIONS


When using your telephone equipment, basis safety precautions should always be followed to reduce the risk of fire, electric shock and injury to individuals.

1. Read and understand all directions.
2. Follow all warnings and instructions.
3. Unplug this product from the wall outlet before cleaning. Do not use liquid cleaners or aerosol cleaners. Use a damp cloth for cleaning.
4. Do not use this product near water. (i.e. Near a bathtub, kitchen sink, washing machine, in a wet basement or near a swimming pool.) Avoid spilling liquid on the product.
5. Do not place this product on an unstable cart, stand or table. Serious damage my occur to this product if it falls.
6. Do not allow anything to rest on the telephone cord. Do not locate this product where the cord might be damaged by persons walking on it.
7. Never put foreign objects of any kind into the product as they may touch dangerous voltage points or short out parts that could result in a risk of fire or electrical shock.
8. To reduce the risk of electric shock, do not disassemble this product, take it to a qualified service person if service or repair work is required. Opening or removing cover may expose you to dangerous voltages or other risks. Operate after incorrect reassembly can cause electric shock.
9. Adjust only the controls that are covered by the operation instructions. Improper adjustment of controls may result in malfunction that could require extensive work by a qualified technician.
10. Refer servicing to qualified service personnel if the following should occur:
 - i. If the phone line or plug is damaged or frayed.
 - ii. If liquid has been spilled into the product.
 - iii. If the product has been exposed to rain or water.
 - iv. If the product does not operate normally by following operating instructions.
 - v. If the product has been dropped or the cabinet has been damaged.
 - vi. If the product exhibits a distinct change in performance.
11. Avoid using a telephone during an electrical storm. There may be risks of electrical shock from lightning.
12. Do not use the telephone in the vicinity of a gas leak.
13. The handset receiver may retain dangerous materials such as metallic pins and staples because the receiver is magnetic.

WALL-MOUNTING

This phone can be wall-mounted if preferred. When wall-mounting the phone you must change the position of the wall-mount-hook via the steps below.

1. Depress and hold down the Hook Switch.
2. Remove the Wall-Mount Hook by sliding upwards
3. Rotate 180 degrees and reinsert. The handset will now place securely on the Wall-Mount Hook.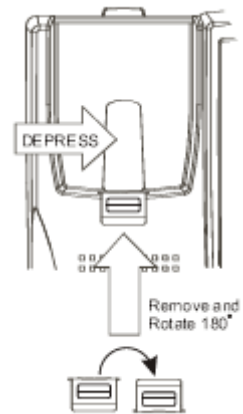

WALL-MOUNTING THE PHONE

1. Choose a location close to the telephone wall socket.
2. Draw marks 83 mm vertically apart on the wall (you can use figure 2 as a guide).
3. Be sure to check your marks before drilling for any wiring or hidden pipes, etc.
4. If it is safe to do so, drill the wall and insert screws, leaving the heads a few millimetres protruded above the wall (figure 3)


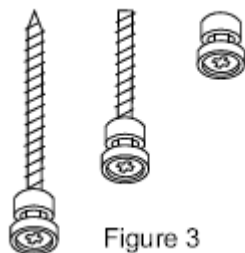
Figure 3

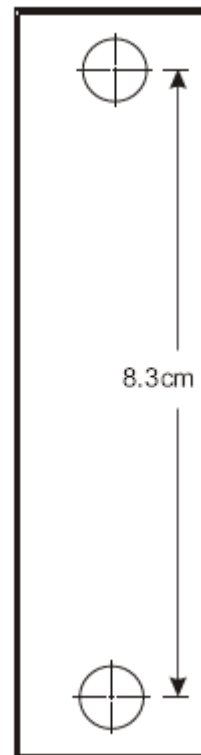SERVICE AND GUARANTEE

Multi Care Systems provides a comprehensive guarantee for two years after the date of delivery. This covers all manufacturing faults. Faults caused by careless use or opening the product are not covered by the guarantee.

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